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



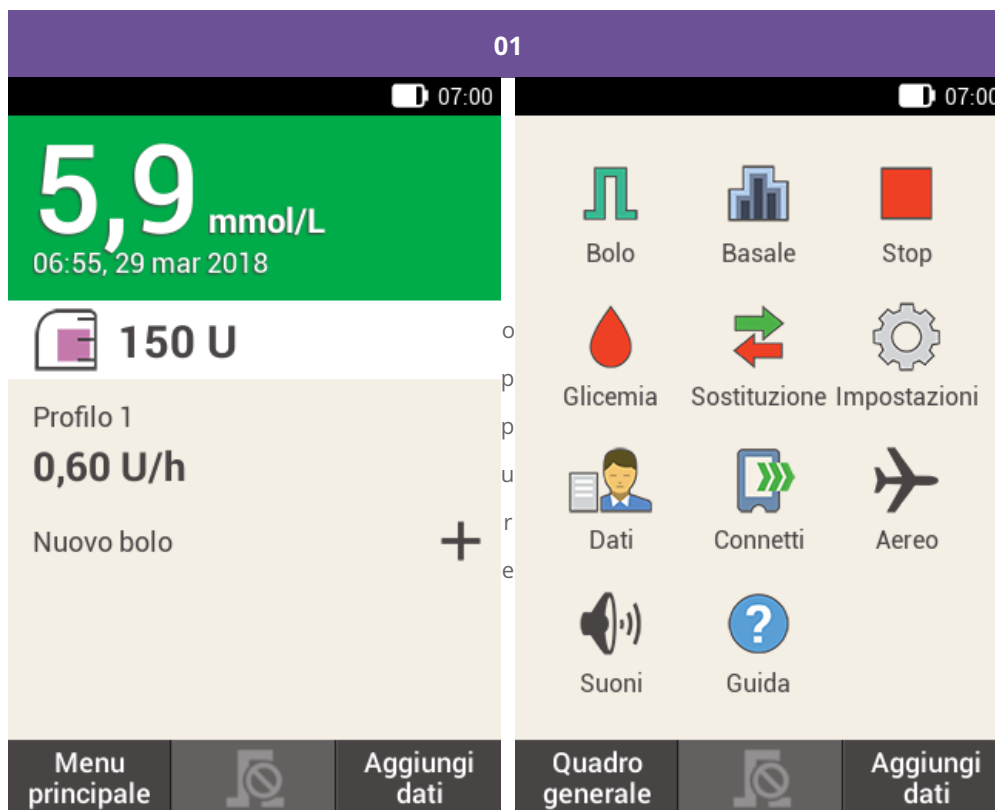
The image shows two side-by-side screenshots of an insulin pump application interface. The left screenshot shows the main menu with a large green box displaying a glucose reading of 5,9 mmol/L at 06:55 on 29 mar 2018. Below this, it shows a bolus of 150 U and a basal rate of 0,60 U/h for Profile 1. The right screenshot shows a grid of icons for various functions: Bolo, Basale, Stop, Glicemia, Sostituzione, Impostazioni, Dati, Connetti, Aereo, Suoni, and Guida. At the bottom of both screenshots are navigation buttons: 'Menu principale' and 'Aggiungi dati' on the left, and 'Quadro generale' and 'Aggiungi dati' on the right.

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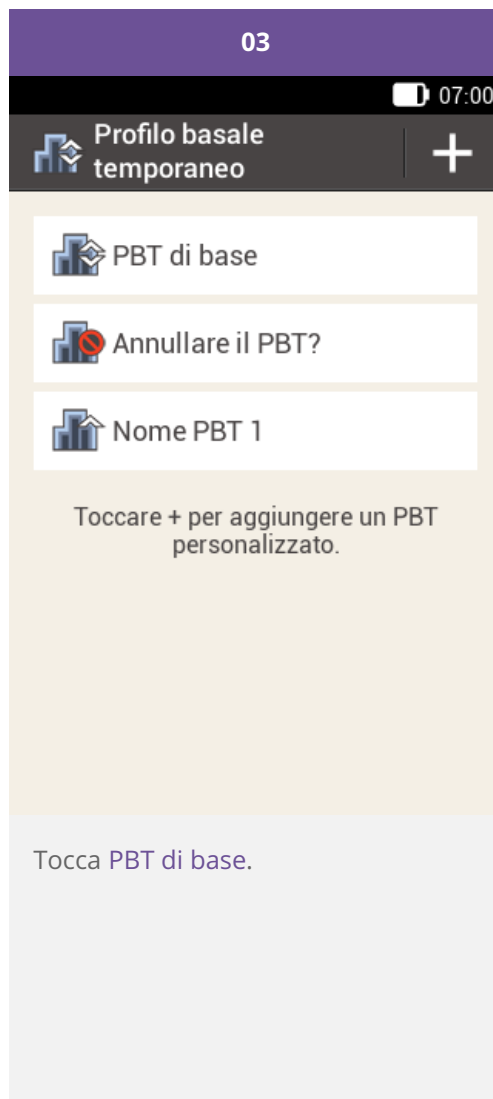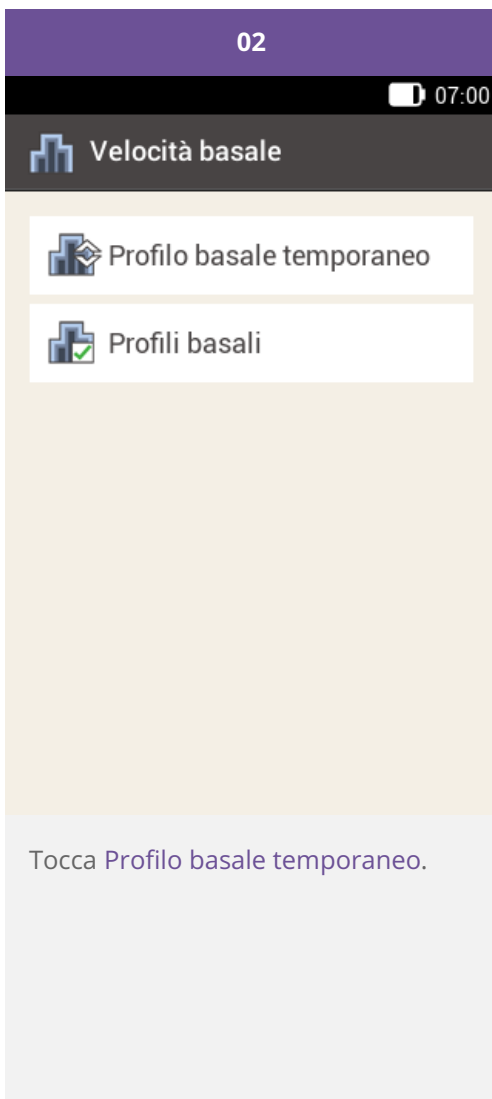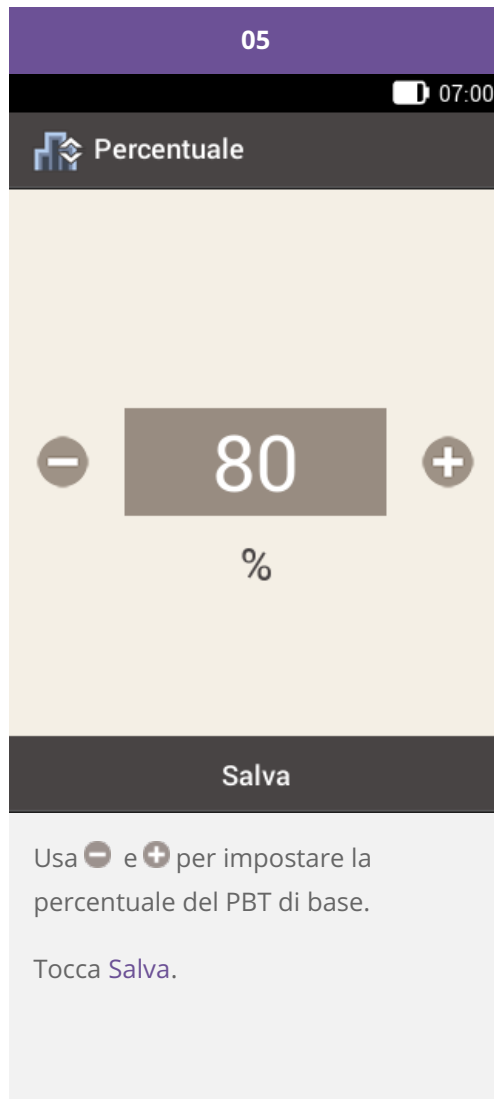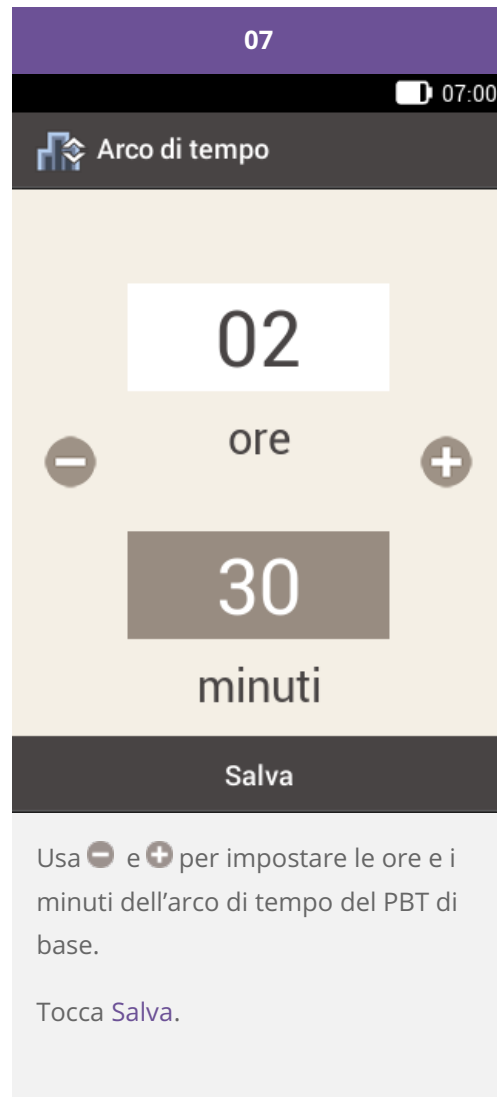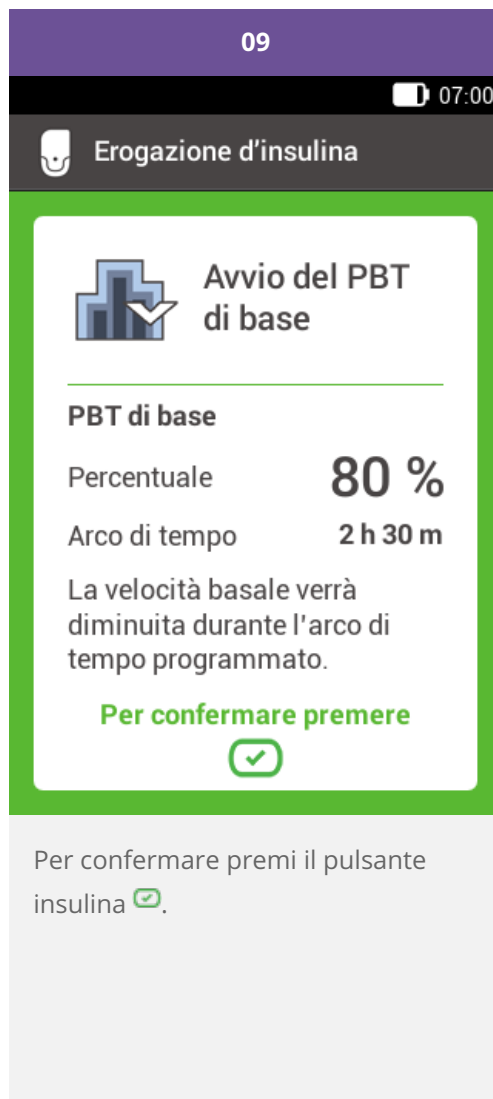
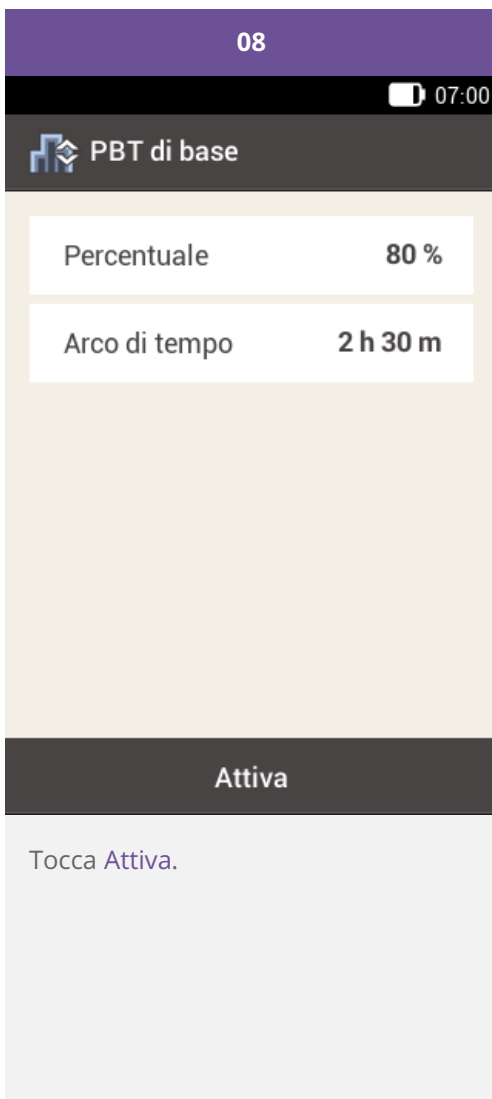
The image displays two side-by-side screenshots of a mobile application interface for insulin pump management. The top status bar shows the time as 07:00 and a battery icon. The left screenshot shows the main menu with a large green display showing a glucose level of 5,9 mmol/L at 06:55 on 29 mar 2018. Below this, it shows a bolus of 150 U and profile 1 with a rate of 0,60 U/h. A 'Nuovo bolo' button with a plus sign is visible. The right screenshot shows a general dashboard with a grid of icons: Bolo (insulin bolus), Basale (basal rate), Stop (red square), Glicemia (blood drop), Sostituzione (replacement), Impostazioni (gear), Dati (person icon), Connetti (connect), Aereo (airplane), Suoni (speaker), and Guida (question mark). At the bottom of both screenshots are navigation buttons: 'Menu principale' and 'Aggiungi dati' on the left, and 'Quadro generale' and 'Aggiungi dati' on the right.

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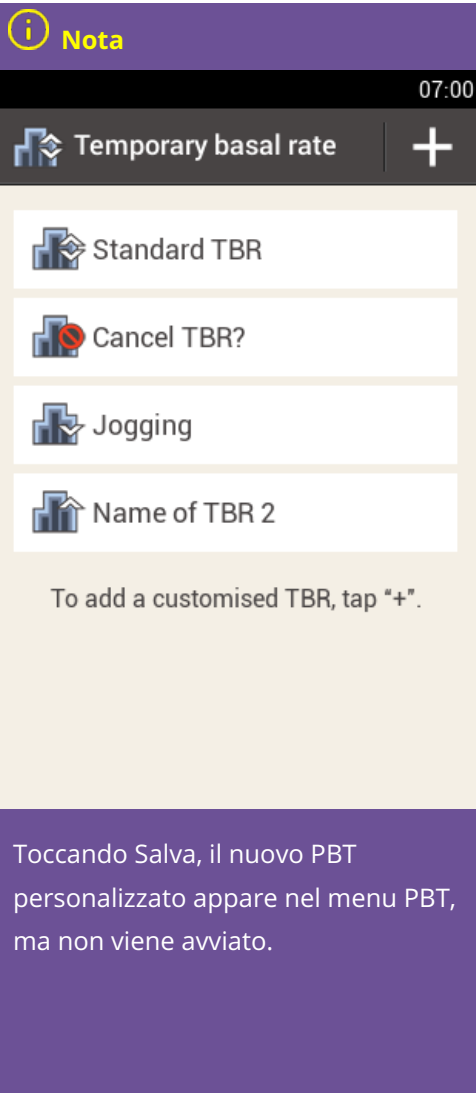
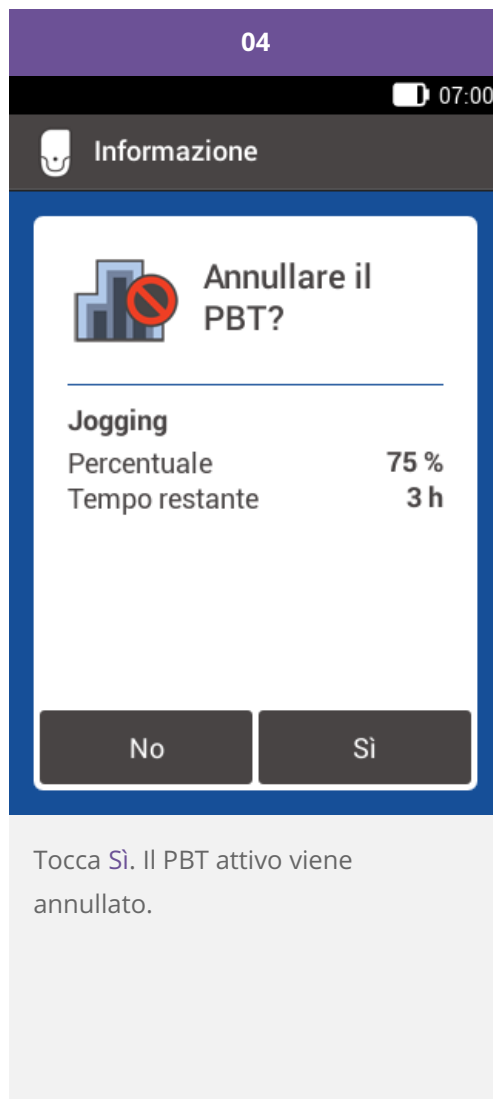
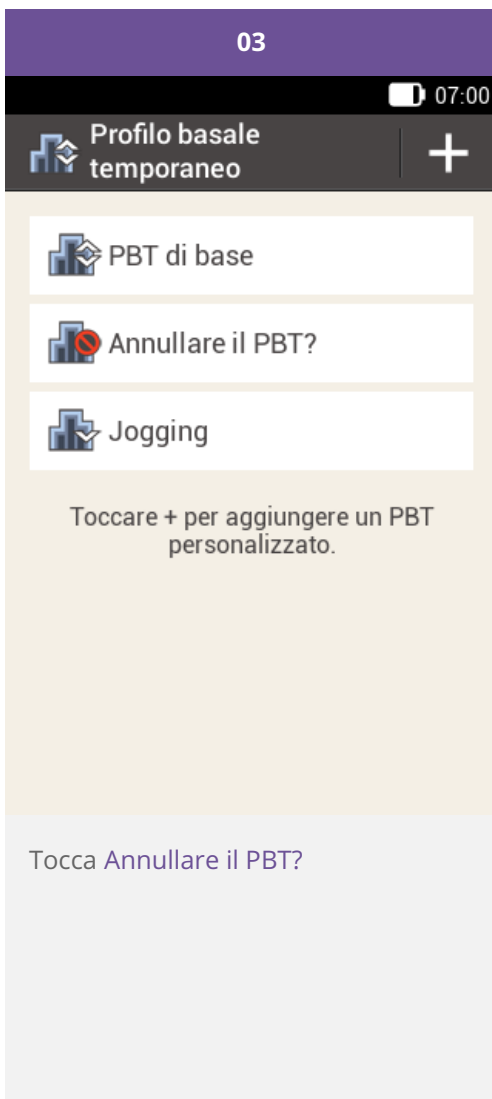
## Annullamento di un PBT

01	02
 <p>The screenshot shows the main menu of the application. At the top, there is a purple header with the number '01' and a black status bar with a battery icon and the time '07:00'. Below the status bar, there is a grid of icons: 'Bolo' (green pill), 'Basale' (blue bar chart), 'Stop' (red square), 'Glicemia' (red drop), 'Sostituzione' (green and red arrows), 'Impostazioni' (grey gear), 'Dati' (person icon), 'Connetti' (blue and green arrows), 'Aereo' (black airplane), 'Suoni' (black speaker), and 'Guida' (blue circle with a question mark). At the bottom, there is a dark grey bar with three buttons: 'Quadro generale', a crossed-out icon, and 'Aggiungi dati'. The 'Basale' icon is highlighted with a white border.</p>	 <p>The screenshot shows the 'Velocità basale' menu. At the top, there is a purple header with the number '02' and a black status bar with a battery icon and the time '07:00'. Below the status bar, there is a dark grey header with the text 'Velocità basale' and a bar chart icon. Below the header, there is a light beige background with two white boxes: 'Profilo basale temporaneo' (with a bar chart icon and a crossed-out icon) and 'Profili basali' (with a bar chart icon and a checkmark icon). The 'Profilo basale temporaneo' box is highlighted with a white border.</p>
<p>Nel menu principale tocca <b>Basale</b>.</p>	<p>Tocca <b>Profilo basale temporaneo</b>.</p>

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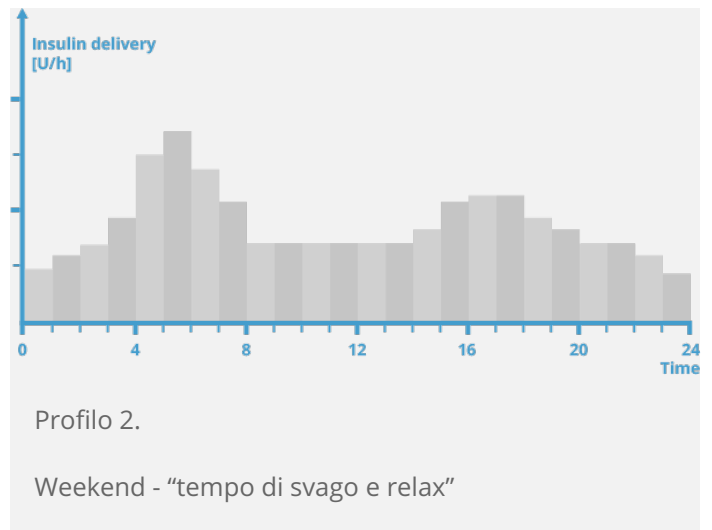
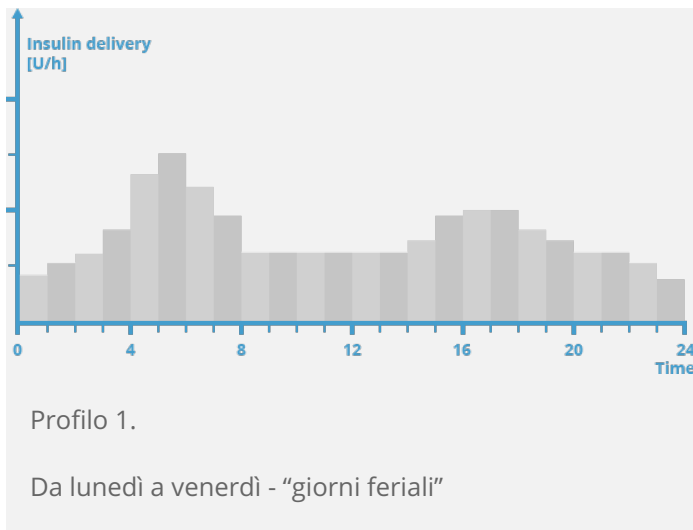
The image shows two side-by-side screenshots of a mobile application interface. The top status bar is purple with the number '01' in the center. Below it is a black bar with a battery icon and the time '07:00'. The left screenshot shows a green header with a large '5,9 mmol/L' and the time '06:55, 29 mar 2018'. Below this is a card with a purple icon and '150 U', followed by 'Profilo 1' and '1,20 U/h'. At the bottom of this card is 'Nuovo bolo' and a plus sign. The right screenshot shows a grid of icons: 'Bolo' (insulin syringe), 'Basale' (bar chart), 'Stop' (red square), 'Glicemia' (red drop), 'Sostituzione' (green arrows), 'Impostazioni' (gear), 'Dati' (person icon), 'Connetti' (blue arrows), 'Aereo' (airplane), 'Suoni' (speaker), and 'Guida' (question mark). At the bottom of both screenshots are three buttons: 'Menu principale', a lock icon, and 'Aggiungi dati'. The left screenshot has a vertical label 'o p p u r e' on the right side of the main content area.

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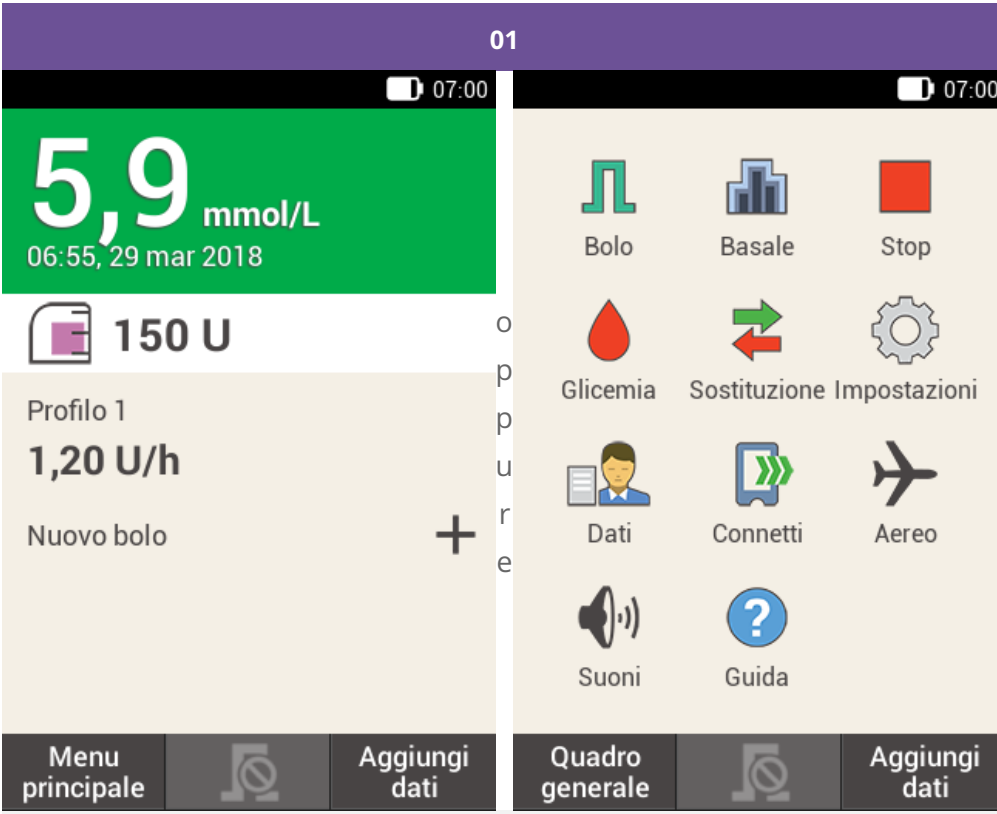
The image shows two side-by-side screenshots of a mobile application interface. The top status bar shows the number '01' and the time '07:00'. The left screenshot displays a large green box with the glucose reading '5,9 mmol/L' and the time '06:55, 29 mar 2018'. Below this, it shows '150 U' with a syringe icon, 'Profilo 1' with a basal rate of '1,20 U/h', and 'Nuovo bolo' with a plus sign. The bottom navigation bar has 'Menu principale' and 'Aggiungi dati'. The right screenshot shows a grid of icons: 'Bolo' (syringe), 'Basale' (bar chart), 'Stop' (red square), 'Glicemia' (red drop), 'Sostituzione' (green arrows), 'Impostazioni' (gear), 'Dati' (person icon), 'Connetti' (blue arrows), 'Aereo' (airplane), 'Suoni' (speaker), and 'Guida' (question mark). The bottom navigation bar has 'Quadro generale' and 'Aggiungi dati'. Vertical text 'o p p u r e' is visible between the two screenshots.

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© 2021 Roche Diabetes Care
Inc. e l'uso di tali marchi da parte di Roche è concesso in licenza.
Roche Diabetes Care GmbH